



E M E R A L D
L A S E R

Checklist

01

Ensure patient is a qualified candidate for Emerald, discuss package options, sign accountability contract [if applicable]

☐

02

Body composition analysis [if applicable], pre-measurements, and before pictures are completed

☐

03

Make sure treatment area is exposed [treatment should be on bare skin]

☐

04

Place center diodes over area of greatest fat accumulation

☐

05

Ensure laser diodes are 3-6 inches away from the surface of the skin [use tape measure to spot check]

☐

06

Turn off any laser diodes in areas not wanting to be treated BEFORE hitting 'START' button

☐

07

When laser pauses at halfway point have patient flip over, re-position laser diodes and spot check distance

☐

08

Upon patient completing treatment ensure they understand aftercare recommendations

☐

09

Give patient checklist handout for aftercare recommendations

☐

10

Progress measurements taken midway through package, final measurements taken, after pictures taken and compiled for patient review

☐